



VEPpeak

Portable device for registering visual evoked potentials

DEVICE CONNECTION



Setup

1. Connect the device cable to the USB port at PC
2. When plugged into a USB port that was not used yet, wait until the device installation is completed
3. If the device is okay, a short flash appears at the stimulator
4. Run the program
5. Press the button **Connect** (if the USB connection is okay, in the right lower part of the program you will see **USB Device connected**)

Connecting the examined person

6. Put the device on the examined person, connect the reference and noise electrode to the ear lobe, or connect also freeelectrodes
7. Due to the dry installation of electrodes it is useful to wait (for about 3 minutes) or dampen the electrodes with water



Examination

8. Set up examination parameters (see the program control description) according to the *examination protocol*, which is specified for each project individually
9. Instruct the examined person to fulfill the task specified in the *examination protocol*
10. Use the button **Record** to perform the recording and save it with the button **Save**
11. Perform all required examinations

Termination

12. To disconnect the device press the button **Close**. If the command is executed successfully, in the right part of the program you will see **Device can be removed**.
13. Disconnect the USB cable of the device from PC
14. Terminate the program

